# Easy-Fit® Jumper

**GB** *Mounting guidelines*

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  **CE 0297** 93/42/EWG

1 patient kit slot .018" – **Oder-no. 321-0118**

1 patient kit slot .022" – **Oder-no. 321-0122**

 **FORESTADENT**<sup>®</sup>  
GERMAN PRECISION IN ORTHODONTICS

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### **Disclaimer**

The Forestadent Bernhard Förster GmbH product guarantee covers material or manufacturing faults only. The operator bears sole responsibility for correct handling and use of products. The Forestadent guarantee does not cover defects that occur due to inadequate treatment planning, incorrect fabrication or use of the treatment appliance.

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## Easy-Fit® Jumper

GB

The Easy-Fit® Jumper is an orthodontic aid from the fixed functional group of appliances for advancement of the mandible in relation to the maxilla. Like the protrusion hinge designed by Prof. Herbst, the Easy-Fit® Jumper is retained in the molar region of the maxilla and the premolar region of the mandible. Unlike the rigid version, the Easy-Fit® Jumper is not retained on bands or cast splints, but in conjunction with standard multibracket treatment by connecting it to the archwire via a rectangular tube.

The following provides some instructions that should be strictly adhered to during fitting.

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## Easy-Fit® Jumper

GB

Fitted Easy-Fit® Jumper with aesthetic Quicklear brackets from teeth 13 to 23, Quick brackets and Quick molar adhesive brackets. Courses offered by Forestadent provide support for the correct fabrication of the appliance. We recommend attending these courses. It is important to follow the stages in the instructions for use exactly when fabricating the appliance.

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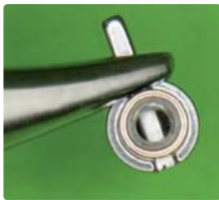
Rod  
Führungsborn

1

**GB** Shorten the rectangular tube (pivot) mesially and/ or distally according to the interbracket distance available: between teeth 7 and 6 in the maxilla, between teeth 3 and 4 in the mandible.

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2

**GB** Attach the pivot to a stainless steel wire dimensions .018" x .025" and place accordingly (slot-filling wire).

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**GB** Place the lower archwire in the multibracket system (place the pivot between teeth 3 and 4), close or ligate the brackets and cinch back distally to secure the arch.

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**GB** Firmly crimp the rectangular tube mesially and/ or distally using blunt side cutters. The pivots should not touch the mesial bracket.

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**GB** Provide additional protection against mesial drift of the pivot using a figure-eight ligature. Alternatively attach an elastic chain.

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**GB** Position the rectangular tube and insert the upper arch, close or ligate the brackets and crimp the rectangular tube. Place the guide tube over the pivot and secure in the pivot with the Allen retention screw. **Ensure that the retention screws are fixed securely in position. The screws can be fixed in position using standard techniques.**

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**GB** The length of the guide tube can be individually adjusted using the key supplied.

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**GB** Widen the eyelet on the rod using a cutter to convert it to a freely moveable hinge. It may be an advantage to bend the eyelet slightly due to the curvature in the canine/ premolar region.

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GB

Insert the rod and secure to the lower arch with the screw.

**Ensure that the retention screws are fixed securely in position. The screws can be fixed in position using standard techniques, e.g. by applying composite.**

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**GB** Check again that the appliance is correctly fitted and make a final adjustment to the planned protrusion – adjust according to the midline!

Anterior view of the finished Easy-Fit® Jumper appliance.

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GB

After fitting:

- Protrusive/ opening exercises should be carried out with the patient.
- The patient should be fully informed about the risks with fixed appliances.
- The patient should be informed about possible disengagement of tube and rod.
- The patient should be instructed on the necessity of avoiding sticky/ hard foods.

Allow an acclimatisation period of one week. Check for correct adaptation no later than two weeks after fitting.

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